

# Aero PLUS

# DISPOSABLE ECG ELECTRODES



# AERO PLUS

## DISPOSABLE ECG ELECTRODE

### PRODUCT USPs :

AREO PLUS ECG electrode provides excellent adhesion with Japanese adhesive, our silver/silver chloride coating and conductive adhesive gel provide an electrode that meets or exceeds international standard. Air tight zip-lock packing provides excellent stability of pack after opening.

### PACKING DETAILS :

500 Pcs in inner Box (50 Pcs x 10 Pouches)  
Master carton of 9 inner boxes = 4500 Pcs

**MRP : Rs. 2400 / 50 Pcs**

Contact Us :

Office (INDIA) : **Hemant Surgical Industries Ltd.**

502, 6th Floor, Ecstasy, Citi Of Joy Commercial Park,  
JSD Road, Mulund West, Mumbai - 400080.

T : +91 22 25912747 | E : [info@hemantsurgical.com](mailto:info@hemantsurgical.com)

W : [www.hemantsurgical.com](http://www.hemantsurgical.com)



# Aero PLUS




## DISPOSABLE ECG ELECTRODES

### ECG ELECTRODES

ई सी जी इलेक्ट्रोड  
ઇ સી જી ઇલેક્ટ્રોડ્સ  
ای سی جی الیکٹروڈس

மின் சி மின்முனைகள்  
ই সি জি ইলেক্ট্রোড  
ഇ സി ജി ഇലക്ട്രോഡുകൾ  
ਈ ਸੀ ਜੀ ਇਲੈਕਟ੍ਰੋਡਸ

*Usage: Application for ECG monitoring or diagnosis with the connected equipment as medical sensors.*

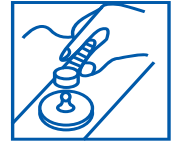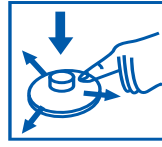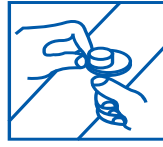

### INSTRUCTION AS ABOVE



1. Select and prepare an electrode site according to your healthcare facility's protocol.
2. Remove electrode from the liner. Apply to skin at standard placement sites
3. Press electrode to skin around outer edge
4. Continue to move pressure towards the center for best adhesion.
5. Attach lead wire to the electrode, start ECG PROCEDURE.

### CAUTIONS

1. Only for hospital or physician.
2. Disposable device, do not reuse.
3. Keep any unused electrodes in the original recloseable pouch.
4. Recommended storage temperature : 5 - 30° C (41-86° F)
5. Electrode use not recommended for excessive 72 Hrs.
6. ECG Electrode must be removed before the patient undergoes X-Ray or MRI scan.



READ INSTRUCTIONS BEFORE USE

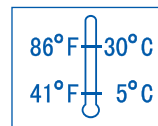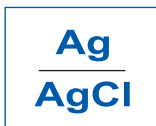

Mfg. and Marketed by :

**HEMANT SURGICAL INDUSTRIES LTD.**



ECSTASY, Office No. 502, 6th Floor, City of Joy Commercial, J.S.D. Road, Mulund (West), Mumbai - 400 080. • Tel. : 91-22-2591 2747, 2591 5289  
Website : [www.hemantsurgical.com](http://www.hemantsurgical.com) • E-mail : [info@hemantsurgical.com](mailto:info@hemantsurgical.com)

**An ISO 9001:2015 Company**